

Trap-Eze™ and Trap-Eze SS (Odor Control)

Disposable Plaster Trap Instructions



Trap-Eze and Trap-Eze (SS) Odor Control Disposable Plaster Traps are designed for containment and disposal of sink waste that has settled out of the drain water. They do not filter but rather increase the diameter of the drain pipe which slows down the drain water flow and allows debris to settle out of the water to the bottom of the trap. **Trap-Eze SS Trap buckets include a time-release odor control material and dispenser permanently sealed to the inside of the bucket lid which is designed to automatically activate upon first use and keep the trap smelling fresh and clean for months.**

Trap-Eze and Trap-Eze SS kits and refills are interchangeable and are available in two sizes. The 3 1/2 gallon standard size is designed to fit most sub-sink areas. The 5 gallon larger size is suitable for dental offices and labs with greater waste needs and where space permits. When selecting trap size, allow 7" to 9" clearance between the Sink Tail Pipe and the top of the trap to ensure the proper positioning of the hoses. *Hoses should have as vertical a slope as possible!* The flexible hoses are designed to be identical and interchangeable so conversion from one trap size to another can be made easily.

Note: Please read these instructions carefully and completely before using this product. Always wear protective eyewear and appropriate safety wear when installing or maintaining this product. It is the responsibility of the user to ensure the product and installation meet all applicable federal, state and local plumbing codes.

Contents

Each Complete Kit includes:

- 1 ea. 3 1/2 Gallon (11" H x 12 1/2" D) or 5 Gallon (14 1/2" H x 12 1/2" D) Disposable Trap (w/ Sealed Top)
- 1 ea. Female PVC Adapter
- 2 ea. 18" long Flexible Sink and Drain Hoses (Ends are 1 1/2" ID (1 3/4" OD))

Each Refill Kit includes:

- 1 ea. 3 1/2 or 5 Gallon Disposable Trap (w/ Sealed Top)
- 2 ea. Red Trap Sealing Caps

Installation

1) Remove the "P" or "U" Trap beneath the sink so that the Sink Tail Pipe and Sewer Tail Pipe are disconnected (see Figure 1.) Use an appropriate plastic sealant to thread the Female PVC Adapter onto the Sewer Tail Pipe to form a tightly sealed connection. Firmly connect one of the Hoses to the Sink Tail Pipe, and the other to the PVC Adapter (see Figure 2.) The Hoses connect by means of firm hand pressure slip fit and friction retention, eliminating the need for tools. Place a small amount of liquid, bar soap or water on the tailstock (see Figure A), then slide the tailstock as far as it will go into the pipe portion of the lid. This will allow tailstock to project below the inside surface of the lid (see Figure B).

2) Position the disposable plaster trap under the sink and fit the hose ends into the white rubber gasket opening in the trap. **Ensure the hose ends are firmly pushed down at least 1" into the gasket openings! Do not cut hose ends or modify in any way!** Applying soapy water to the mating parts before connection will enhance the fit and removal of the trap. Partially filling trap with water during hook up will help stabilize the trap.

IMPORTANT- Do not constrict hoses or excessively confine trap in a sub-sink area. Rotate the fully connected trap to enhance proper vertical slope. Test trap after installation to detect any faulty connections. Notify Buffalo Dental immediately to report problems or defective parts. **Do not stand on trap. Always turn off sink faucet when unattended or not in use.**

Maintenance and Trap Replacement

We recommend that you change the trap regularly, once every 6 to 12 months (or more frequently for heavy usage). Check trap weekly. Change the trap when residue accumulates to within 2" of the top of the trap. Be sure to have a spare refill ready at all times. To replace the trap, disconnect the hose from the sink tail pipe first and dispose of excess water. Seal the full container with the provided red end caps and discard in accordance with local regulations. For the regular Trap-Eze, to help maintain sanitary conditions it is recommended to flush a couple of spoonfuls of granulated chlorine powder into the trap once a week. This may be purchased at any pool supply store (should not be required for the Trap-Eze SS under conditions of normal use).

Caution: Should any solvents (i.e., wax removers) be washed into the trap, be sure to flush thoroughly to prevent these materials from floating within the trap.

Ordering Information

Item #	Description	Item #	Description
62100	Trap-Eze 3 1/2 Gallon Complete Kit	Sanitrap 1	Trap-Eze SS 3 1/2 Gallon Complete Kit
62106	Trap-Eze 3 1/2 Gallon Refill	Sanitrap 2	Trap-Eze SS 3 1/2 Gallon Refill
62110	Trap-Eze 5 Gallon Complete Kit	Sanitrap 3	Trap-Eze SS 5 Gallon Complete Kit
62112	Trap-Eze 5 Gallon Kit	Sanitrap 4	Trap-Eze SS 5 Gallon Refill
62108	Hoses Only, Set of 2 (18" long each, Hose Ends are 1 1/2" ID, 1 3/4" OD)		

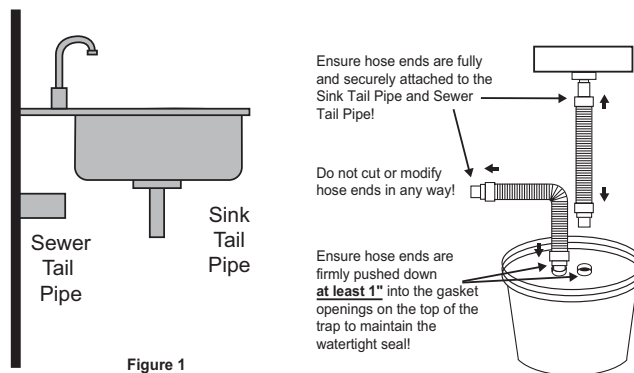


Figure 1

Figure 2



FIGURE A

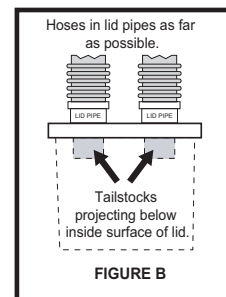


FIGURE B

Buffalo Dental Manufacturing Co., Inc.

159 Lafayette Dr., PO Box 678, Syosset, NY 11791-0678 | Tel: 516.496.7200 or 800.828.0203

Fax: 516.496.7751 | Email: sales@mailbdm.com | www.buffalodental.com